


Nepus(NE-SM1)

Cleaning info




STEP 1
Quick
Cleaning
(After use)

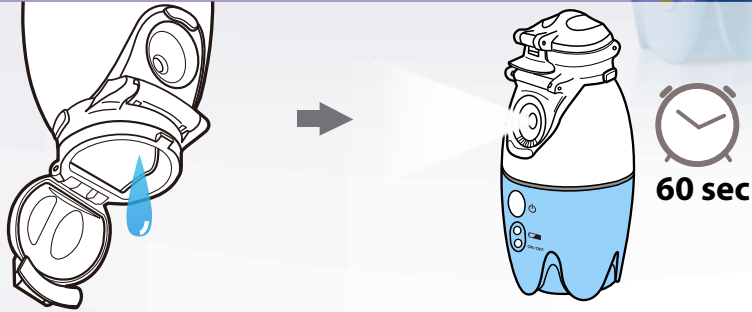

1) After filling with water until half full, gently shake it. Dispose of residual medication.

2) Nebulize for 60 sec after filling with clean water. Warm water is better.


STEP 2
Accessories
Cleaning
(After use)

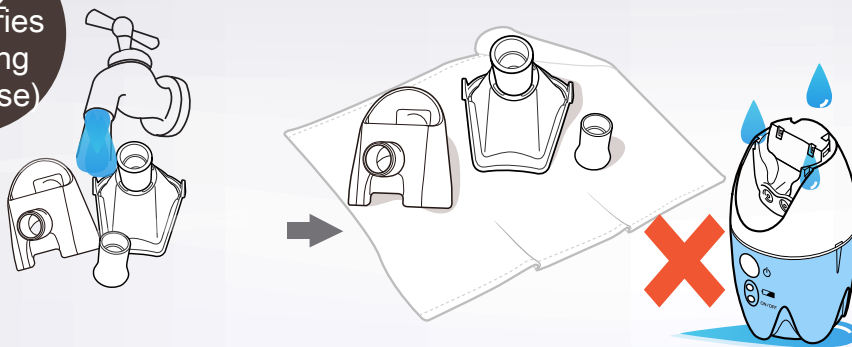

1) Gently shake the parts in running water

2) Right after cleaning, remove water with gauze (paper towels) and dry them completely.


Cleaning
(If necessary)

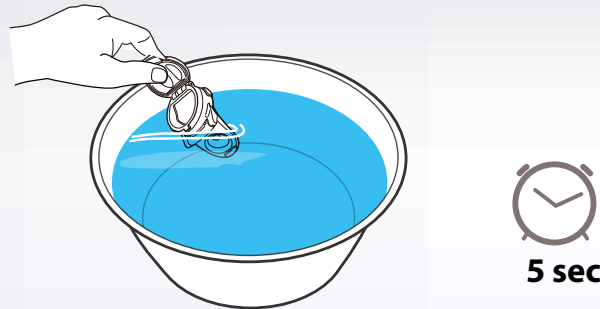

1) Gently shake medication bottle (mesh cartridge) in warm water or room temperature water for 5 seconds.

2) Right after cleaning, remove water with gauze (paper towels) and dry it completely.

Accessories
Disinfecting
(If necessary)

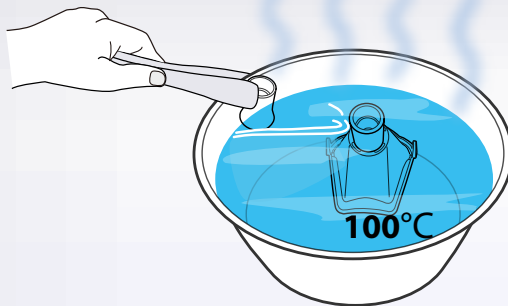

1) Disinfect the parts including mask by dipping in hot water (100°C) for 1 minute.


2) Right after disinfecting, remove water with gauze (paper towels) and dry them completely.

Neplus(NE-SM1)


Caution




STEP 1
Quick
Cleaning
(After use)

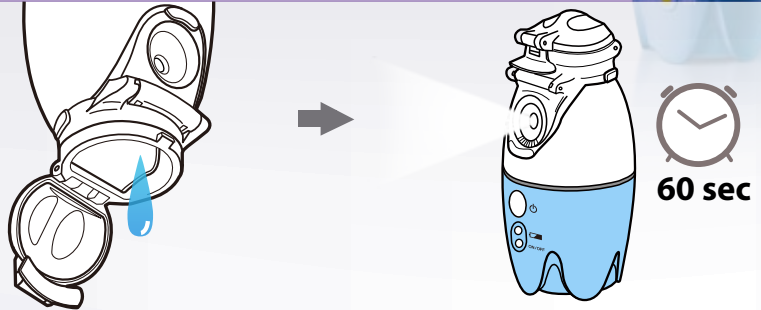


STEP 2
Accessories
Cleaning
(After use)

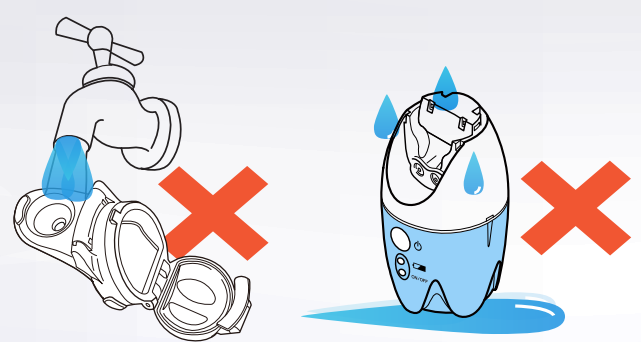

Cleaning
(If necessary)

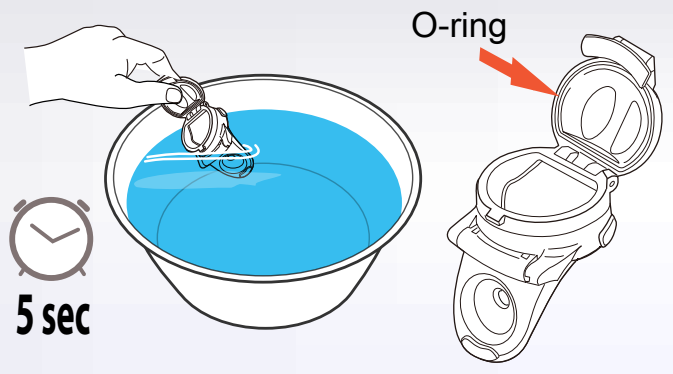

Correct position
of Battery cover

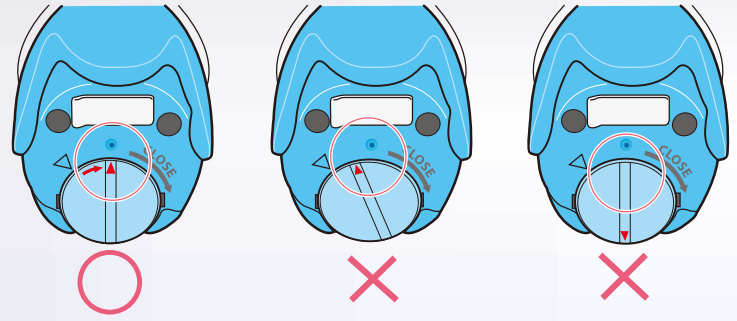

- Never use a disinfection solution including hypochlorite solution.
- For cleaning, use only clean water.
- Do not clean medication bottle (mesh cartridge) in running water.
- Do not clean the main unit with water.
- Do not put the main unit(**battery cover**) in wet place.
- Frequent cleaning may cause damage of the part.
- O-ring of medication bottle cap could be missing during cleaning.
- Do not dip over 5 seconds.
- Wrong position of battery cover causes the unit not to work.
- Check proper battery polarity when inserting one.